


**Instructions to have your WristJack remanufactured:**

Order catalog no. CFD-147-R  
Agee WristJack External Fixation System, REMANUFACTURED  
Price: \$825.00  
Allow 4 weeks for remanufacturing and return

Note: The remanufactured WristJack will be a **NON-STERILE, SINGLE USE** device that must be steam sterilized. It will come complete with instruments, pins and instructions.

 **Hand Biomechanics Lab**  
77 Scripps Drive, Suite 104  
Sacramento, CA 95825  
(800) 522-5778  
(916) 920-2215 Fax

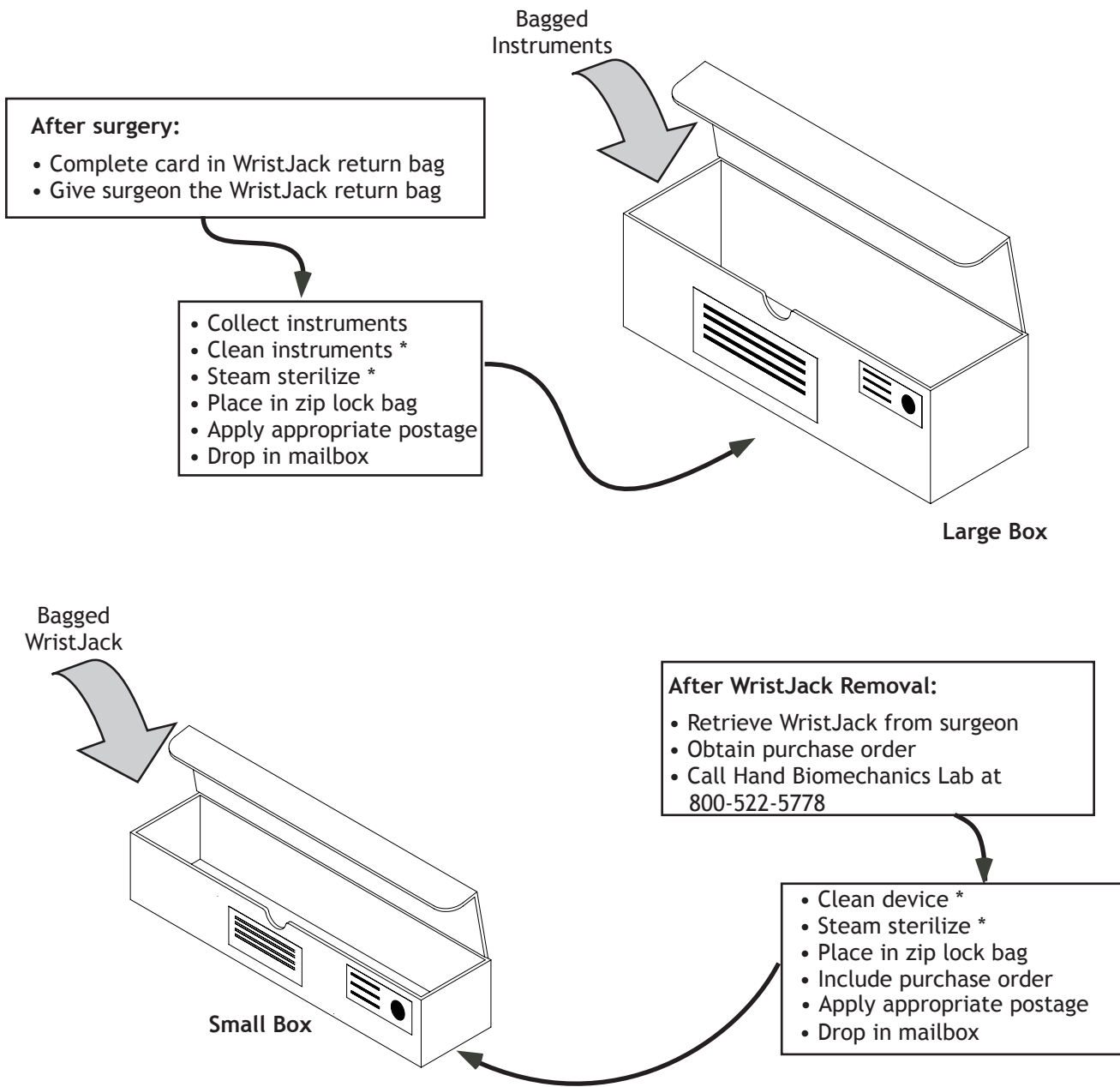

\* Note: Clean and sterilize the instruments and WristJack before returning at follows: Remove adherent visible soil from the device and/or instruments using detergent, water and manual processes where necessary. Then steam sterilize (flash or wrapped) in a pre-vacuum sterilizer at 270-272 degrees Fahrenheit for 4 minutes.

## **Instructions to have your WristJack remanufactured on reverse side.**

Hand Biomechanics Lab, Inc. is the original manufacturer of the WristJack Fracture Reduction System. This flyer describes our service to remanufacture your used WristJack Systems. The process starts with you returning the instrument after surgery, then the WristJack after patient removal. We will disassemble, clean and inspect the returned WristJack and instruments. Wear sensitive parts are replaced, then the device is reassembled and fully tested to the same standards as a new device. The WristJack, instruments, new fixation pins and new surgeon's instructions are then repackaged in NON-STERILE form and sent back to you. You will need to steam sterilize before use. The remanufactured WristJack remains a SINGLE USE device.