



Hand Biomechanics Lab

Endoscopic Carpal Tunnel Release Disposable Blade Assembly

Indications for Use

- For use with the 3M Agee Inside Job Carpal Tunnel Release System or the MicroAire SmartRelease ECTR System in minimally invasive ligament release of the carpal tunnel in the wrist.

Contraindications

- Refer to your 3M or MicroAire Instructions for Use.
- Not compatible with any other ECTR systems.

Warnings and Cautions

- Refer to your 3M or MicroAire Instructions for Use.
- Sterile. Single use only. Do not re-sterilize. Do not use if package is open or damaged.

STERILE R



Instructions for Use

- Have adequate training and understanding of the associated anatomy as well as use and applications of your 3M or MicroAire system.
- Follow your current 3M or MicroAire Instructions for Use, substituting this device for the MicroAire Blade Assembly.
- Ensure the Blade Assembly is secured with the Lock Screw after insertion into the handpiece.

The terms "3M" and "3M Agee Inside Job Carpal Tunnel Release System" are trademarks of 3M Company. The terms "MicroAire" and "SmartRelease" are trademarks of MicroAire Surgical Instruments, LLC.

PN 345200